


COAGUCHEK SET UP PROCESS

Step 1:

Log in to INR Online

Step 2:

Go to More>CoaguChek Link

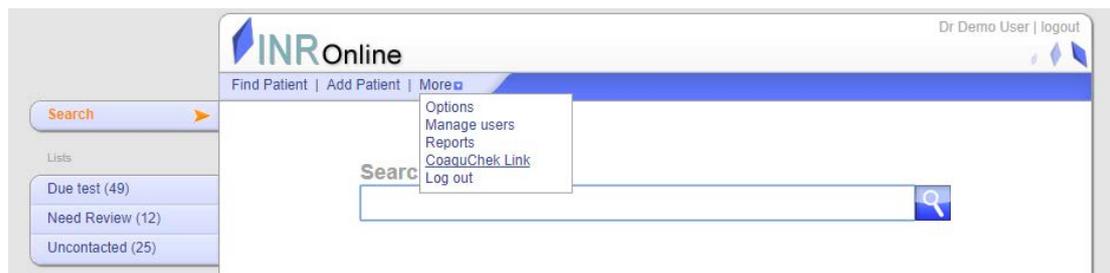

Step 3:

Click on CoaguChek Link and you will be taken to the Download page.

HealthObs Connect

Step 1: Installing HealthObs Connect

The HealthObs Connect program allows INR Online to link directly to a CoaguChek XS Plus, with the Handheld Base Unit.

To install HealthObs Connect, download and run the install file by clicking the below button.



Advanced

The drivers for the CoaguChek should automatically be installed with HealthObs Connect. However if you experience a problem you can download them here. [CoaguChek drivers](#)


[Show advanced setup.](#)

Step 4:

Click "Download"

When you click "Download" the HealthObs Connect software should be the first to load. Depending on the browser you are using, you may have to "Allow" or "Run Anyway" to allow the software to install. We recommend Chrome as your browser.

This pop up box should appear.

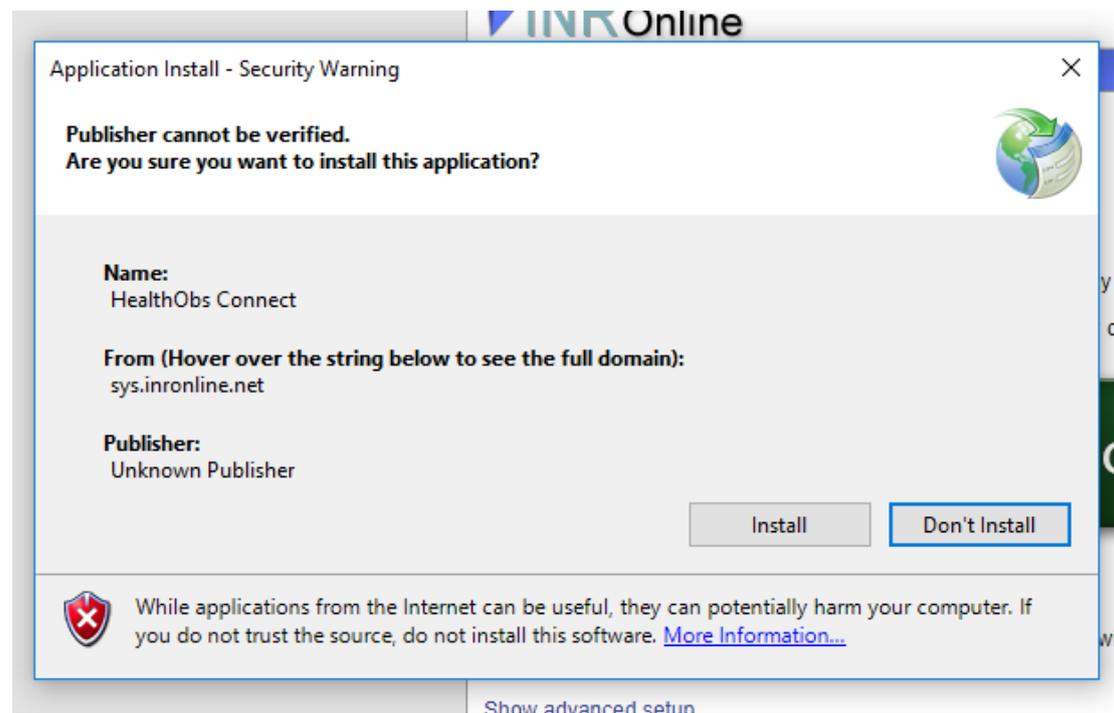

Step 5:

Select “Install”

The Download page on the website should look like this

HealthObs Connect

Step 1: Installing HealthObs Connect

The HealthObs Connect program allows INR Online to link directly to a CoaguChek XS Plus, with the Handheld Base Unit.


To install HealthObs Connect, download and run the install file by clicking the below button.



Advanced

The drivers for the CoaguChek should automatically be installed with HealthObs Connect. However if you experience a problem you can download them [here](#). [CoaguChek drivers](#)

The Handheld Base Unit software should automatically start to install. If it doesn't, go to the Hidden Icons and look for the HealthObs Connect Icon.

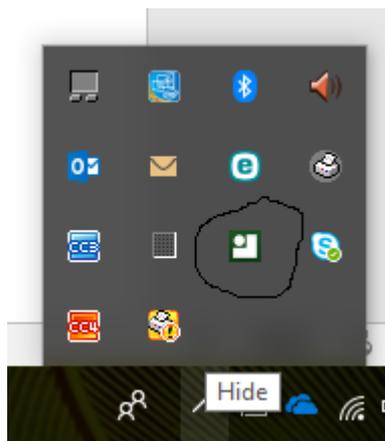


Right click on the icon and select “Setup”. This should start the Handheld Base Unit software installation.

Make sure the CoaguChek IS NOT connected to the computer


You may have to “Allow” this installation to proceed.

Step 6:

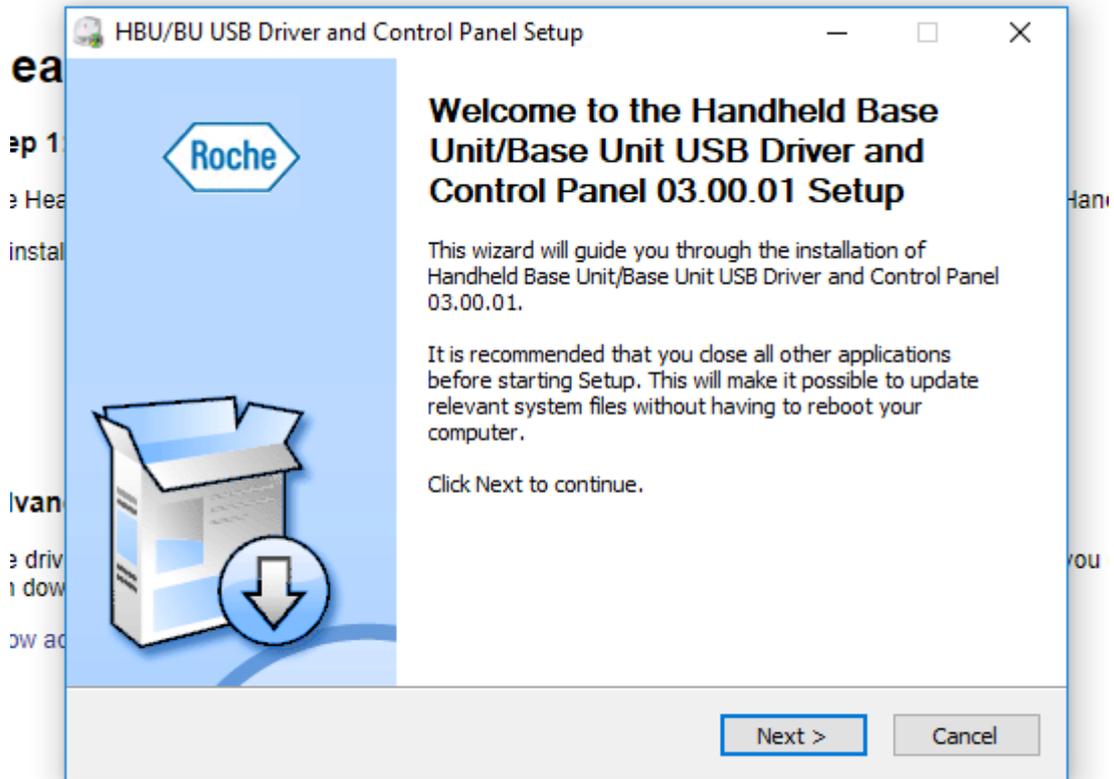


Select "Next"

Step 7:

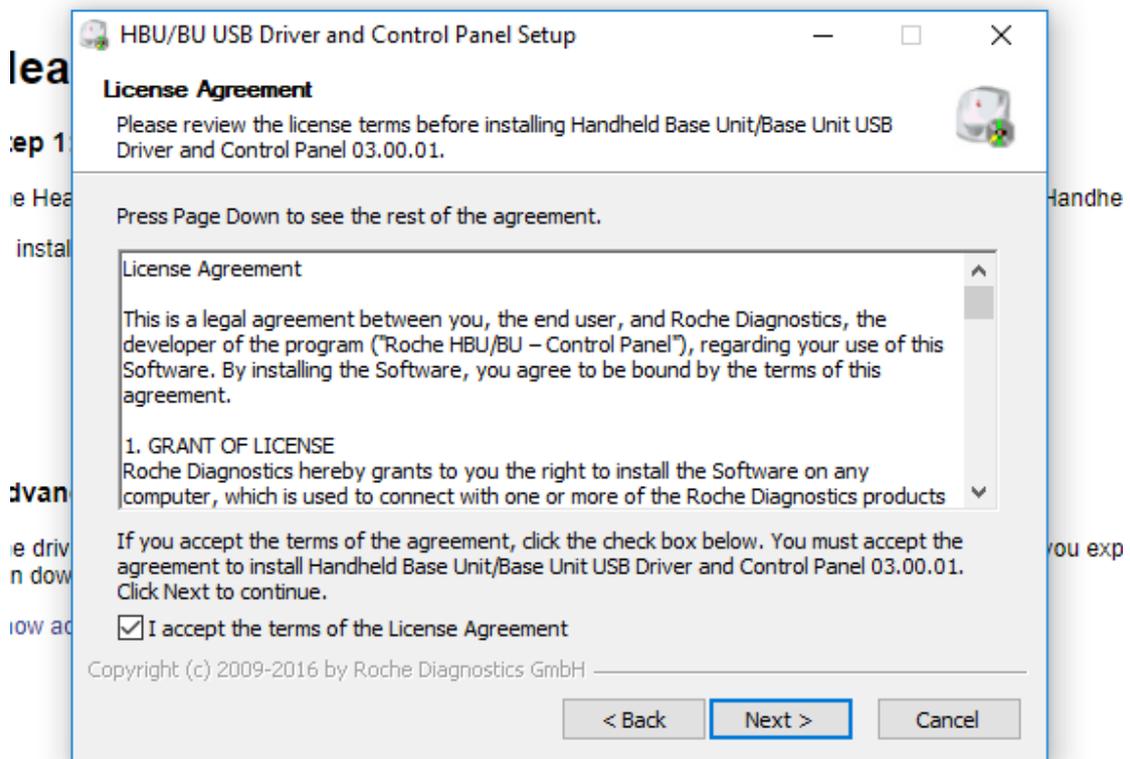


Accept the terms of the License Agreement and select "Next"

Step 8:

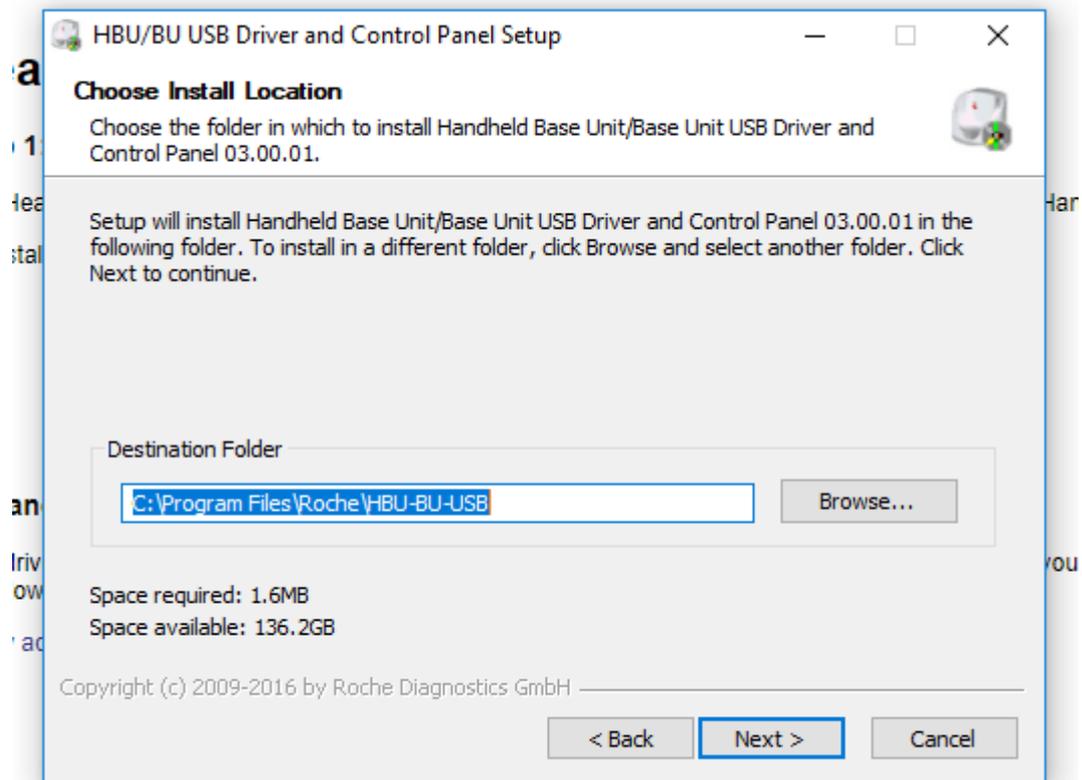


Select "Next"

Step 9:

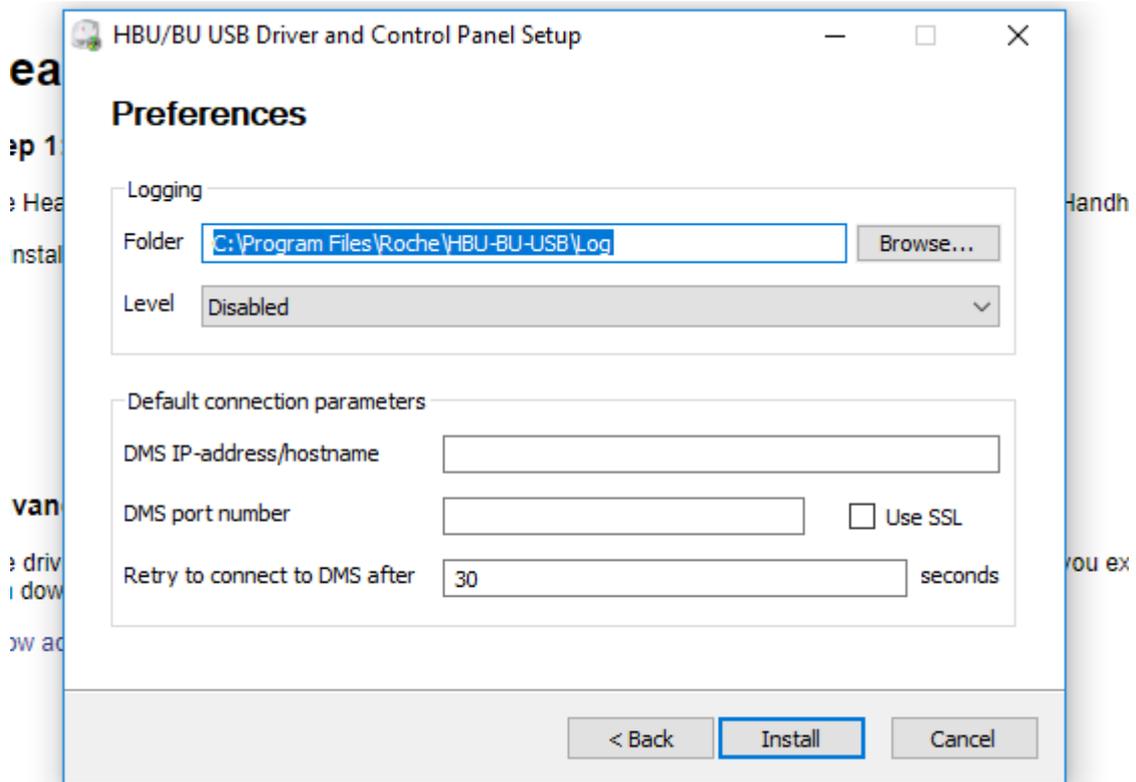


Select "Install". You do not need to fill in any of the fields.

Step 10:

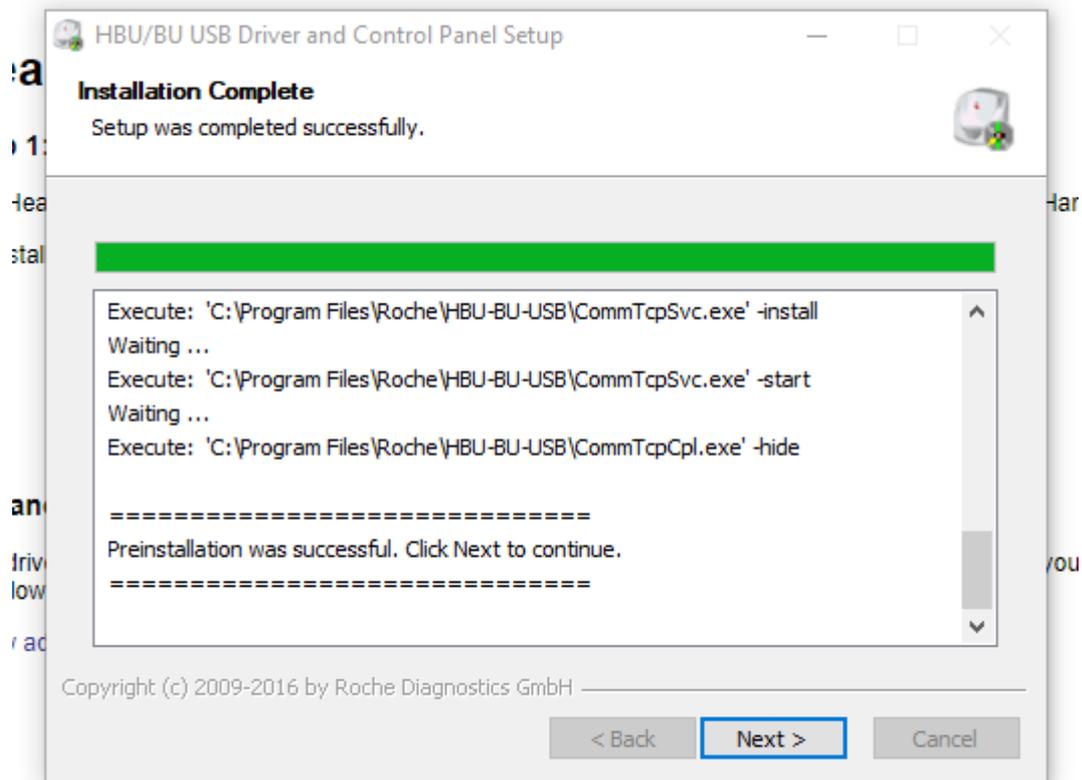

Click "Next"

Step 11:

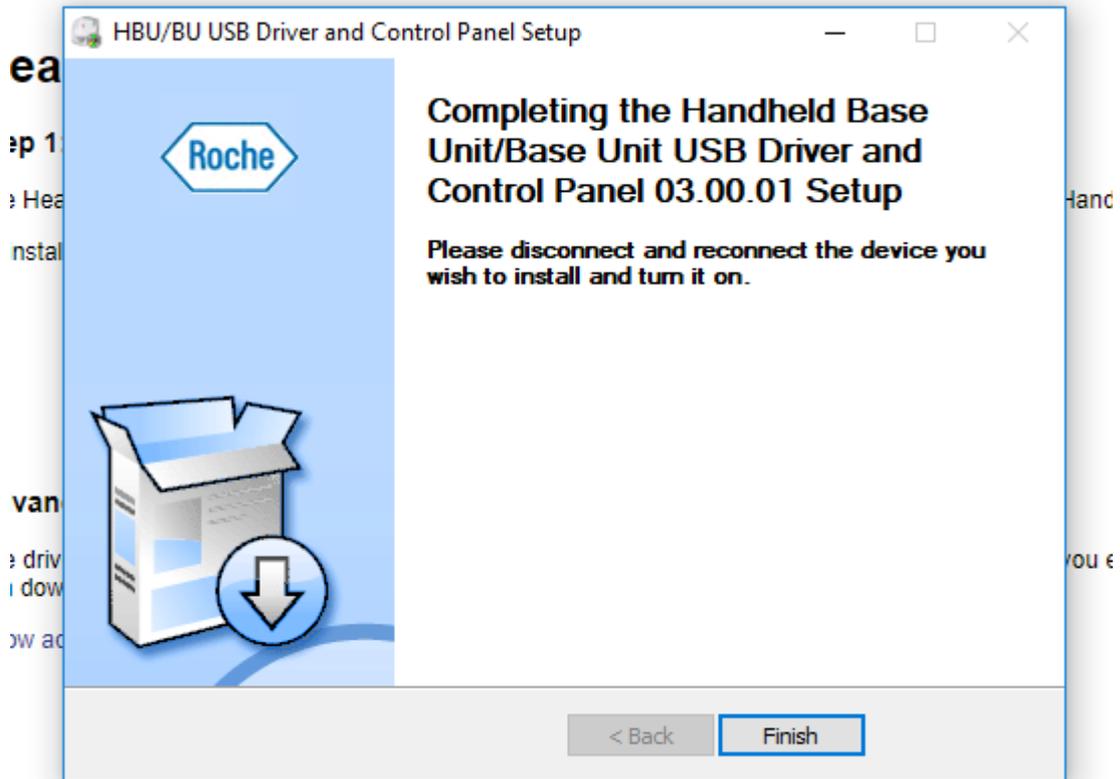

Select "Finish"

Step 12:




Connect the CoaguChek to the computer and select "Next"

Step 13:



The HealthObs Connect software will now look for the CoaguChek

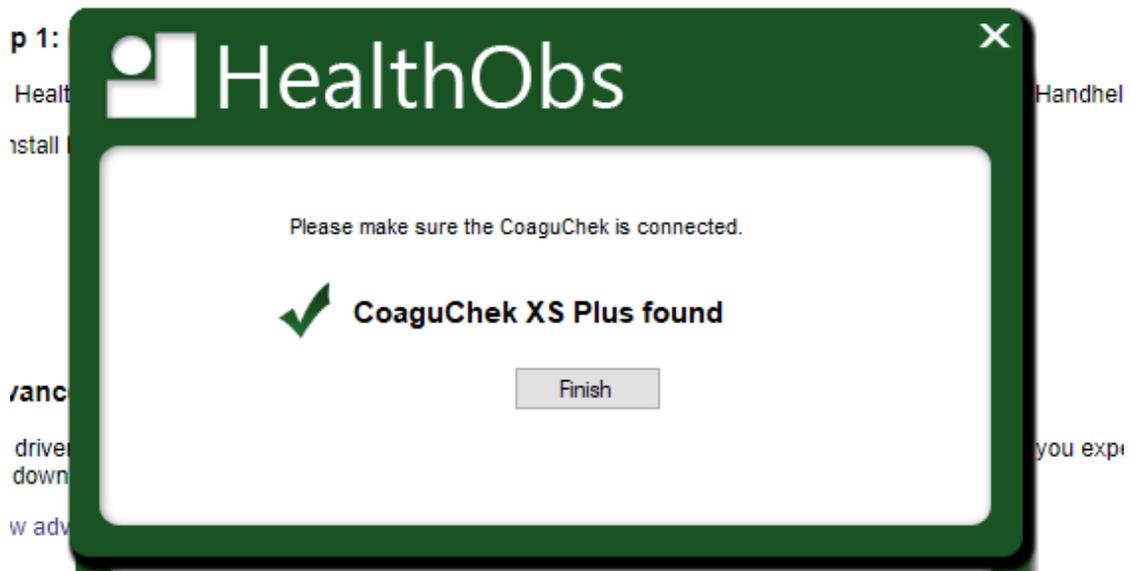

The CoaguChek has been found. Select "Finish"